

DELMAR HYPERTONIC nasal spray

Medical device

Instruction for use

DELMAR HYPERTONIC is a nasal spray with sea salt. The content of salts in the solution is higher than inside cells of the body, salt concentration corresponds to salinity of seawater. Such solutions are called hypertonic. After application to swollen nasal mucosa DELMAR HYPERTONIC removes water by natural mechanism of osmosis, thereby reduces swelling and facilitates breathing through the nose. DELMAR HYPERTONIC is intended for long term use by persons with impaired function of the nasal mucosa, relieves rhinitis and feeling of nasal congestion without causing damage to the nasal mucosa or loss of efficiency.

When to use DELMAR HYPERTONIC

- when you feel stuffy nose
- in acute rhinitis
- when you have rhinitis or rhinitis of allergic origin
- when you have sinusitis
- before use of nasal corticosteroids
- for mitigating the effects of long-term use of nose drops that contain vasoconstrictors
- when you have problems with snoring

How to use DELMAR HYPERTONIC

Adults and children from 2 years of age: Apply one to two sprays into each nostril several times a day as needed.

Before use, remove the protective ring and remove the transparent protective cover of the nozzle. Hold the spray upright with a nozzle upwards during the application. Push downwards the dosing mechanism several times until the solution splashes. Put gently spray nozzle into the nostril and press the dosing mechanism, while you still hold the spray upright and do not lean your head. Inhale through the nose during the splashing gently. After use, wash the nozzle with sufficient amount of water and put the protective cover and ring to its original position. From the hygiene reasons, each user should have the own spray.

More information

Sensitive patients may experience an allergic reaction, which is usually mild and resolves after discontinuation of use. There are not sufficient data to assess the risk of DELMAR use during pregnancy and breast-feeding. Duration of use is not limited.

Composition: purified water, sea salt, benzalkonium chloride. 1 ml of solution contains 37 mg of sea salt and 0.1 mg of benzalkonium chloride.

Storage conditions: Store below 25°C. Keep out of the sight and reach of children.

Shelf life: 3 years from the date of manufacture

Pack size: paper box with a spray of 50 ml (contains approximately 450 doses)

Manufacturer: MEDOCHEMIE Ltd., 1-10 Constantinoupoleos Street, 3011 Limassol, Cyprus

The latest revision of the instructions for use: 2016/10

www.delmar.cz